

Practical and easy to use:

- ▶ Leave the Skin Cooler in the freezer for about 1 hour (until it freezes);
- ▶ Remove and gently glide over the skin for a few minutes, massaging the region to be treated;
- ▶ After use, wash with soap and water and put back in the freezer/freezer to freeze. It can also be disinfected with a degerming solution or alcohol.

Precautions:

- ▶ Keep out of reach of children;
- ▶ Avoid falls;
- ▶ Non-sterilizable product in autoclave;
- ▶ Do not expose to heat, microwave or dishwasher;
- ▶ Do not use abrasive products for cleaning.

Customize with your company:

We have the option of customization with the printing of the logo or company name in the minimum order. Contact us for more information.

- Pain numbing applicator;
- Gel spreader for laser;
- Replaces ice pack;
- Good for massaging;
- Stainless steel roller.

